




Lohmann & Rauscher

SwellSpots™

Oedema and fibrosis pads.



Ideal for difficult areas of oedema, fibrosis, skin folds, misshapened limbs as well as padding and protection.

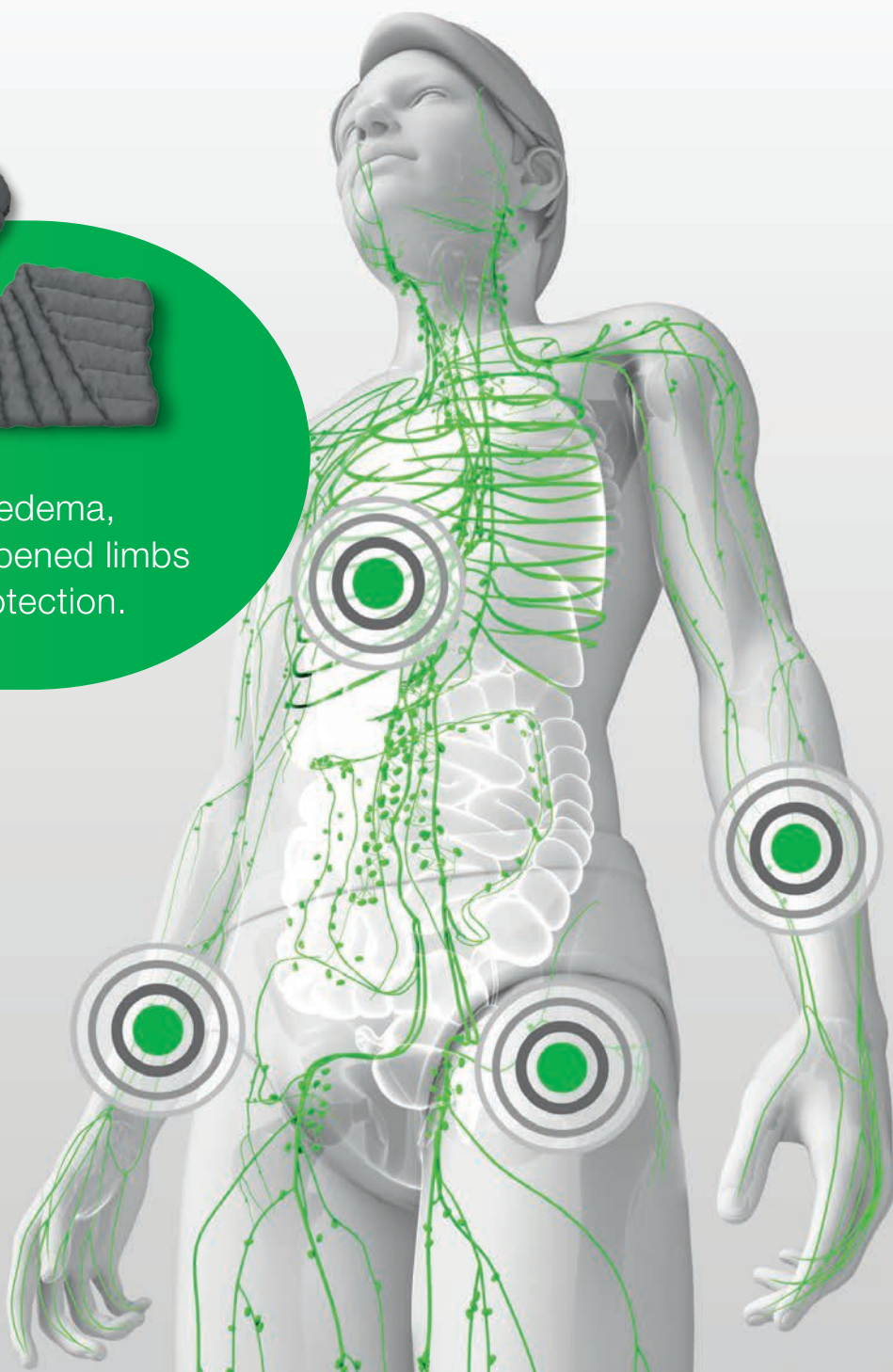

POWERED BY
coolcore®

Overview

Why choose SwellSpots™?

POWERED BY


SwellSpots, oedema and fibrosis pads are designed to address particularly troublesome areas by:

- softening tissue fibrosis/hardened skin
- filling concavities
- padding bony prominences
- diffusing pressure points
- protecting vulnerable areas

SwellSpots are available in a wide range of versatile shapes and sizes that tuck into compression garments or under bandages. Powered by Coolcore, these SwellSpots rapidly wick away sweat and moisture from the body, keeping it cool and comfortable.

What is Coolcore?

Coolcore is an innovative, chemical-free fabric that rapidly wicks away sweat and body moisture to regulate body temperature. Its chemical-free technology is built into the fibres of the fabric to withstand its cooling properties through wash and wear during the one year life of the garment.




88%
faster wicking
sweat and moisture
away from the body*



55%
faster
drying time
for easier
garment care*



30%
more
cooling power
to better regulate
body temperature*

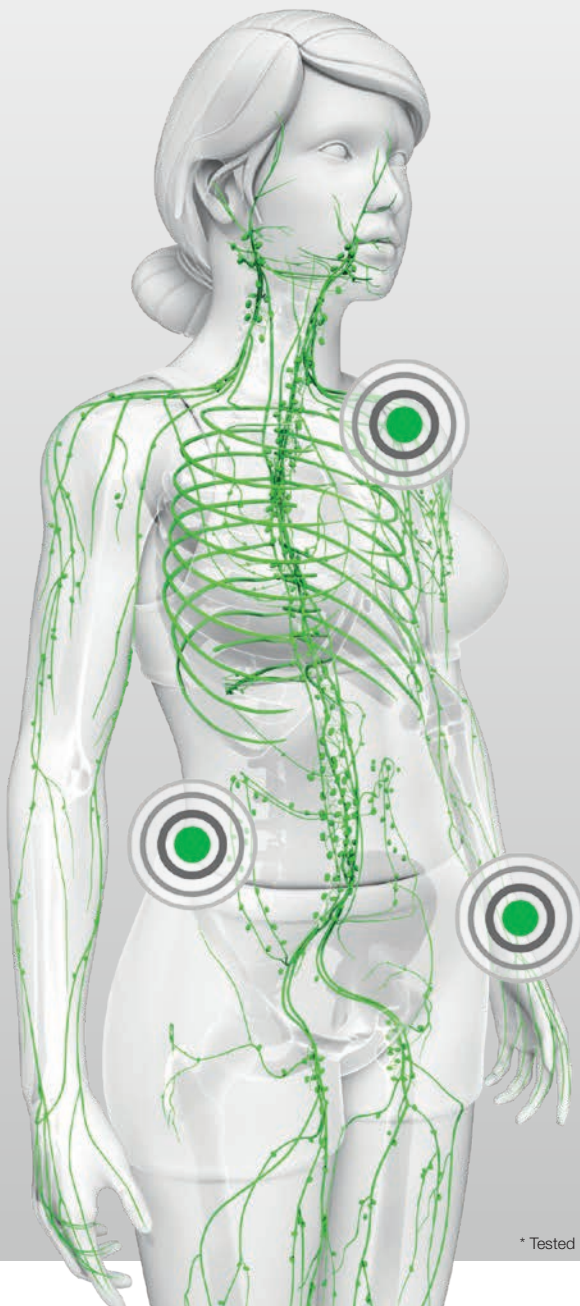


foam technology helps soften
tissue fibrosis and hardened skin



creates multipoint compression
profiles which help encourage
lymphatic reabsorption




versatile shapes and sizes
available off-the-shelf

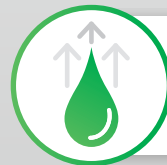

the special Coolcore fibre
helps wick away moisture



saves time from cutting
your own chip bags

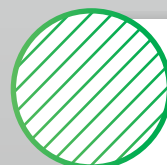

chevron channelling based on
manual lymphatic drainage (MLD)
principles of directing lymphatic
fluid to collateral pathways

* Tested by Hohenstein Institute, an accredited laboratory and research institute focused on testing and certifying textiles.

SwellSpots™



Full Bra Pad


Pad designed for use from the lateral chest wall toward the anterior midline of the chest, and can be worn inside a bra.

Cup size*	REF	Shipping unit
A-B cup	104 556	1/5
B-C cup	104 555	1/5
C-D cup	104 554	1/5

Lateral Bra Pad

Pad designed for use on the lateral chest wall, and can be worn inside a bra.

Width (cm)	Height (cm)	REF	Shipping unit
10	16	104 553	1/5

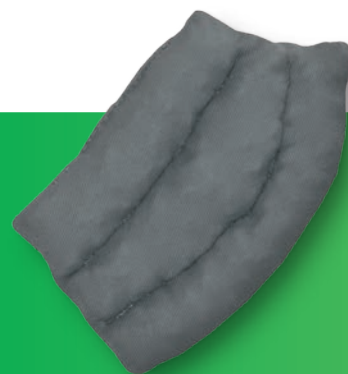

Breast Pad

Pad designed specifically for use in the breast area, and can be worn inside a bra. Its seamless design allows the Breast Pad to be used on either side of the chest.

Cup size*	REF	Shipping unit
A-B cup	104 541	1/5
C-D cup	104 540	1/5
D-DD cup	104 539	1/5



Palmar Pad

Versatile pad designed for use on palmar surface of the hand, dorsum, concave shape of palm and inside a bra.

Width (cm)	Height (cm)	REF	Shipping unit
7	8	104 531	1/5
8	9	104 530	1/5

SwellSpots™



Kidney Pad (previously Kimbe)

Versatile pad designed for use at the scapula, groin and neck.

Width (cm)	Height (cm)	REF	Shipping unit
14	24	104 538	1/5



Circular Pad (previously Sobol Small & Large)

Versatile pad designed for use on the chest wall, hip, anterior knee, posterior knee and abdomen.

Diameter (cm)	REF	Shipping unit
18	104 536	1/5
22	104 535	1/5

Care Instructions for SwellSpots

Washing

- machine-wash with mild detergent on permanent press
- do not use fabric softener, bleach or other additives
- do not dry-clean

Drying

- tumble-dry on permanent press
- do not iron

Materials

- blend of: polyurethane, nylon, polyester, elastane
- not made with natural rubber latex



Indications

- lymphoedema (when used with bandages, e.g. Rosidal K/other compression systems, e.g. ReadyWrap)
- oedema (when used with bandages, e.g. Rosidal K/other compression systems, e.g. ReadyWrap)

Precaution

- allergies or sensitivities to any of the product materials
- care should be taken in patients using anticoagulant medication to assess tolerance

For all queries relating to SwellSpots as well as ReadyWrap, Garesia, TributeNight or Exo products please contact your local distributor:

VELCRO® is a registered trademark of Velcro Industries B.V. Used with permission.

Lohmann & Rauscher Pty Ltd · 321 Ferntree Gully Road · Mount Waverley, Victoria 3149, Australia · Customer Service: 1300 572 869
Lohmann & Rauscher Limited · 21 Queen Street · Auckland 1010 · New Zealand · Customer Service: 0800 572 869